

 **Emperor****EmFilm Post-Op**
Shaped Film Dressing with Absorbent Pad

Featuring a latex-free, semi-permeable polyurethane film membrane which is coated with a hypoallergenic acrylic adhesive, EmFilm Post-Op Dressings are designed to be breathable with good oxygen and moisture vapour exchange.

Waterproof and impermeable to liquids, bacteria and viruses, the risk of infection is reduced, and the dressing also incorporates an integral central pad consisting of absorbent material.



Facilitates wound healing



Provision of a 'moist wound healing environment'



The patient can shower with the dressing in place



Prevention of contamination by dirt or bacteria, thus a reduced risk of wound infection



Rounded edges and consistent hypoallergenic adhesive help prevent dressing damage and early removal, reducing the number of dressing changes needed.

Call: 0800 008 6646

E-mail: woundcare@rociallehealthcare.com

Visit: www.emperorwound.com

 **Rociale**
Healthcare

Emperor is a product brand of Rociale Healthcare

Size	Product Code	NHSSC Code	PIP Code	Drug Tariff Price
10cm x 20cm	537713ZD	ELW85029	424-0072	£1.96
10cm x 25cm	537714ZD	ELW85021	424-0064	£2.00
10cm x 30cm	537715ZD	ELW85020	424-2326	£2.99
10cm x 35cm	537716ZD	ELW85042	424-2334	£3.35

For **FREE** product samples and more information, please get in touch

EmFilm Post-Op Application Guide

Preparation:

1. Use existing clinical guidelines in preparation for the application of EmFilm Post-Op
2. Select a dressing that is of a suitable size and shape. Choose the dressing size large enough to provide a margin that adheres to dry healthy skin around the wound site

Dressing Application:

1. Peel the two white paper liners outwards from the dressing to expose the absorbent pad and adhesive border
2. Position the absorbent dressing pad centrally over the wound
3. Gently press down the adhesive border onto the skin covering the wound with the absorbent pad
4. As indicated by the blue strip remove the transparent release film on the back of the dressing from the centre outwards
5. Smooth the entire dressing from centre towards the edges using gentle pressure to enhance adhesion



Call: 0800 008 6646

E-mail: woundcare@rociallehealthcare.com

Visit: www.emperorwound.com

 **Rociale**
Healthcare

Emperor is a product brand of Rociale Healthcare